



A Clever Solution to Treat Anal Fissure


Maintaining relaxation of the internal anal sphincter muscle is critical to reducing anal fissure pain. Studies have shown that delivering a controlled dose of topical medication such as Diltiazem, Nifedipine or Nitroglycerine within the anal canal (anoderm) leads to greater internal sphincter pressure reduction and significant pain relief.¹

Targeted Delivery of Medication

DoseRite's patented applicator provides a simple and painless way for patients to apply medication inside the anal canal. The applicator's side-slotted design ensures medication is applied directly to the anal mucosa (anoderm) for maximum absorption by the internal sphincter muscle.

DoseRite Provides Consistent Pain Relief

DoseRite helps patients get the relief they need by taking the guesswork out of anal fissure treatment. Using DoseRite three times per day provides prolonged relaxation of the anal sphincter muscle and significant pain relief.¹

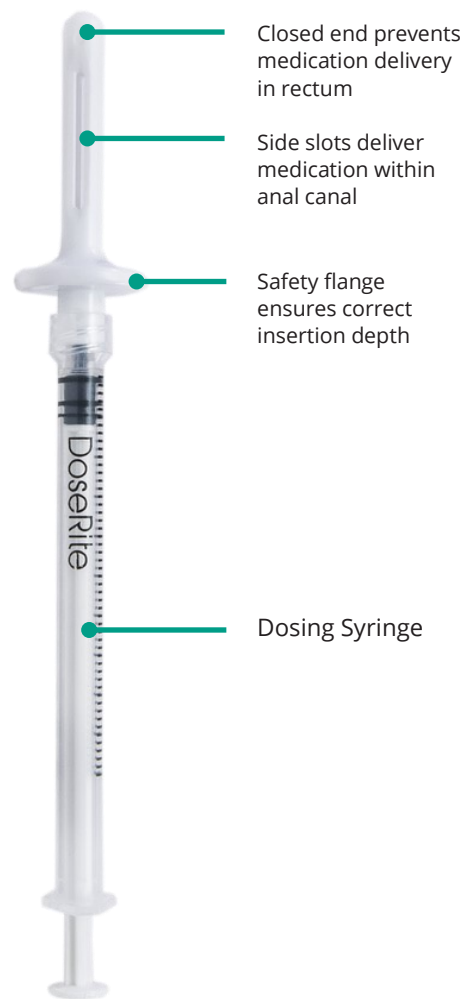



Simple Accurate Dosing

DoseRite dosing syringes connect directly to the patient's medication container. This clever system allows patients to apply an accurate single dose every time for more consistent and predictable results.

DOSING GUIDELINES

(Physician information)

The manufacturer recommends the following:

Calcium Channel Blocker (Diltiazem, Nifedipine)	0.5ml/dose 3 doses/day
Nitroglycerine	0.3ml/dose (equal to 1-inch of medication) 2 doses/day
Approximate medication loss with applicator	0.05ml

Conveniently Packaged for Patients


The DoseRite Anal Fissure Kit includes the following:

- 5 each Dosing Syringes
- 5 each Reusable Applicators
- 1 each Dosing Jar with Syringe Adapter
- 1 each Cleaning Tool and Spatula



Targeted Delivery of Medication

Medication delivered within the anal canal is absorbed more quickly and effectively by the anal mucosa providing prolonged relaxation of the internal sphincter muscle and significant pain relief.

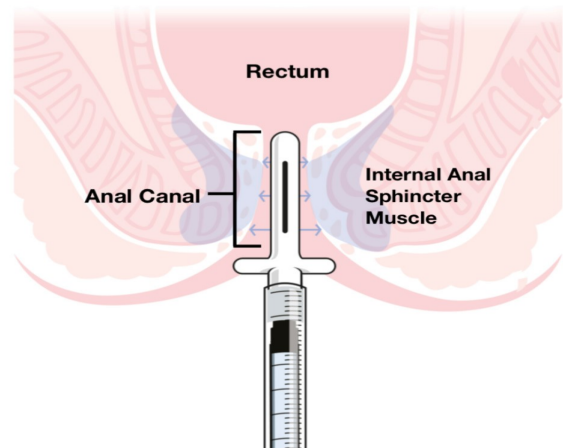

Medication (blue) is delivered within the anal canal for improved absorption by the anoderm and greater relaxation of the internal anal sphincter muscle

Ordering Information

For additional information, or to order this product, please call 01844 340 620 or email info@aprmtech.com.

Code	Description	Box qty
DRJ05AT-6M	DoseRite Reusable Pack	1

Additional product information is available at: aprmtech.com/doserite

Manufactured by:

Surgin

www.doserite.info

apr medtech